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CYTOCARE S Line is a sterile single use medical device class III (CE 0123) manufactured by REVITACARE, indicated to reduce differently ageing effects linked to age, life style, or environment. The Glogau reference has selected to distinguish skin type as mild (I), moderate (II), advance (III) and severe (IV). CYTOCARE S Line improves skin quality for a short duration (filling fine lines, attenuating marked wrinkles, restoration of hydration and radiance, maintaining densification) of skin type (Glogau III and IV). For medical use. Read carefully CYTOCARE S Line Instructions For Use before using the product. CYTOCARE S Line must be injected in an aseptic manner, by duly doctor with a good knowledge of the anatomy and physiology of the area to be treated, with sterile and single use equipment supplied by REVITACARE. Before use CYTOCARE S Line, check the expiry date on the syringe labelling, the integrity of the primary packaging for the syringe and needles and the appearance of the solution. Store between 2°C and 30°C, away from the light in its original secondary packaging. Do not use if package is damaged. Once the syringe of CYTOCARE S Line has been opened, use it immediately. Discard the syringe, needles and the remaining product after use, in accordance with the guidelines in force for their disposal. Single use solution: do not re-sterilise and do not re-use. The doctor must inform the patient that there are potential adverse effects linked to the injection of this device, which occur immediately or are delayed. Adverse events observed with CYTOCARE S Line: - pain and bleeding during the injection - tingling, redness, inflammation, erythema and itching after the injection - bruise, oedema, nodule and swelling at the injection site - infection resulting from incorrect asepsis - low efficacy. The patient must inform the doctor as soon as possible if any other adverse events persist for longer than one week. The doctor must remedy it by an appropriate treatment. The distributor and / or manufacturer must be alerted to any other adverse event related to the injection of CYTOCARE S Line. Other adverse events observed in the literature concerning cross-linked hyaluronic acids or not, these potential risks should be taken into account: - small haematomas and allergic reaction - transient loss of sensitivity in the injected area, colouration or discolouration in the injected area and cases of necrosis in the glabellar region, abscess, granuloma, hypersensitivity and acne papules. This medical device is not reimbursed by health insurance organisation. This medical device is a regulated health product which, under this regulation, bears the CE marking.

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